FICHE TECHNIQUE

ELECTRODES DE DEFIBRILLATION RADIO TRANSPARENTES CADENCE MEDITRACE

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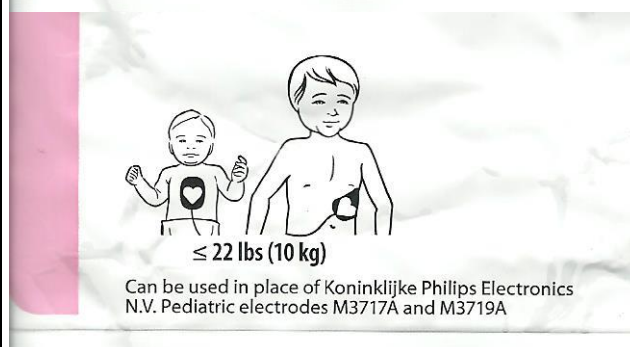
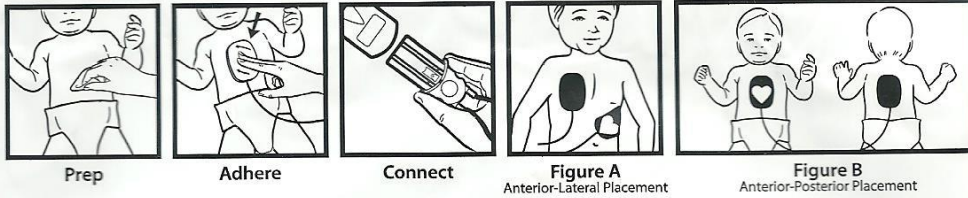
ELECTRODES DE DEFIBRILLATION RADIO TRANSPARENTES CADENCE MEDITRACE

Instructions:

- Clean and dry skin sites. Note: Do not use alcohol or tincture of benzoin.
- Remove electrodes from package. Separate lead wires.
- Remove the electrode protective liner.
- Position electrodes as pictured in Figure A or Figure B. Maintain at least 1 inch (2.5 cm) separation between electrode edges. Do not place on broken skin, if possible. Figure A and Figure B are guidelines for positioning electrodes and are not intended to replace clinical preferences.
- Smooth the electrode from the center outward to the edges with fingertips to ensure that there are no air pockets between the gel and the patient's skin.
- Electrodes are not repositionable. Replace with new electrodes if repositioning is required. This will ensure optimal adhesion and minimize potential patient skin burns.
- Connect the electrodes to the defibrillator.
- Follow the defibrillator instructions.
- Replace the electrode pads after 24 hours on skin, 25 defibrillation shocks, or 1 hour of pacing. Replace electrodes sooner if irritation or burns occur.
- Remove electrode by slowly peeling pad from the patient's skin. Discard used electrodes.

Warnings:

- Place electrodes according to defibrillator manufacturer's operating instructions.
- Use only with manual defibrillators or in manual defibrillation mode. Not for use with Automatic External Defibrillators (AED).
- Do not exceed 100 J while defibrillating. Use local protocol or American Heart Association guidelines to select appropriate defibrillation energy level corresponding to patient's body weight.
- Continuous pacing may cause skin irritation or burns. Inspect skin underlying the electrodes frequently after 30 minutes of continuous pacing and replace electrodes if skin irritation or burns occur. Replace electrodes after cessation of pacing.
- Do not discharge hand-held paddles through these electrodes.
- Do not use if electrode or pouch is damaged.
- Do not open package until immediately prior to use.

Important:

- Operator should be thoroughly familiar with electrode operating instructions prior to use.
- Use separate ECG electrodes when performing noninvasive pacing.
- Do not crush, fold, or store under heavy objects.
- Please contact your Covidien representative for specific model details.

Monophasic or Biphasic defibrillation/Cardioversion/Pacing/ECG Electrode
 Electrode de défibrillation//cardioversion/stimulation/ECG monophasée ou biphasée
 Einphasige oder zweiphasige Defibrillations-/Kardioversion/Schrittmacher-/EKG-Elektrode
 Electrodo per defibrillazione/cardioversione/stimolazione/ECG monofase o bifase
 Desfibrilación monofásica o bifásica/Cardioversión/Estimulación/ECG Electrodo
 Monofasisk eller bifasisk defibrillerings-/elkonvertering/pacing/EKG-elektrod
 Eenfasige of tweefasige defibrillatie-/cardioversie/pacing/ecg-elektrode
 Electrodo de Desfibrilção/cardioversão/Estimulação/ECG Monofásico ou Bifásico
 Yksivaiheinen tai kaksivaiheinen defibrillaatio-/Kardioversio/tahdistus-/EKG-elektrodi
 Elektrode til mono- eller bifasisk defibrillering/Kardioversion/pacing/EKG
 Μονοφασικό ή διφασικό ηλεκτρόδιο απινιδίωσης/καρδιονάταξη/βηματοδότησης/
 ηλεκτροκαρδιογραφικό ηλεκτρόδιο

Can be used in place of Koninklijke Philips Electronics N.V.
 Pediatric electrodes M3717A and M3719A
 Peuvent être utilisées à la place des électrodes pour enfant
 Koninklijke Philips Electronics N.V. M3717A et M3719A
 Kompatibel zu Elektroden für Kinder von Koninklijke Philips
 Electronics N.V., M3717A und M3719A
 Possono essere utilizzati al posto degli elettrodi M3717A e
 M3719A per pediatria Koninklijke Philips Electronics N.V.
 Pueden utilizarse en lugar de los electrodos pediátricos
 M3717A y M3719A de Koninklijke Philips Electronics N.V.
 Kan användas i stället för pediatrika elektroder med nummer
 M3717A och M3719A från Koninklijke Philips Electronics N.V.
 Kunnen worden gebruikt in plaats van Koninklijke Philips
 Electronics N.V. elektroden voor kinderen M3717A en M3719A
 Podem ser utilizados em vez de electrodos pediátricos M3717A
 e M3719A da Koninklijke Philips Electronics N.V.
 Voidaan käyttää Koninklijke Philips Electronics N.V:n
 pediatristen M3717A- ja M3719A-elektrodien sijaan.
 Kan anvendes i stedet for Koninklijke Philips Electronics N.V.
 pædiatriske elektroder M3717A og M3719A
 Μπορούν να χρησιμοποιηθούν στη θέση των παιδιατρικών
 ηλεκτροδίων M3717A και M3719A της Koninklijke Philips
 Electronics N.V.

LOT 206969
 use by 2014-04
 00228 CH42Z100286

U.S. Patents 5,571,165; 5,733,324; 5,824,033; 6,600,957.
 © 2012 Covidien. Made in USA.
 Covidien LLC, 15 Hampshire Street, Mansfield, MA 02048 USA.
 Covidien Ireland Limited, IDA Business & Technology Park, Tullamore.

FICHE TECHNIQUE

ELECTRODES DE DEFIBRILLATION RADIO TRANSPARENTES CADENCE MEDITRACE

 COVIDIEN™

REF 22660R

Medi-Trace™ Cadence

Adult Multi-Function Defibrillation Electrodes

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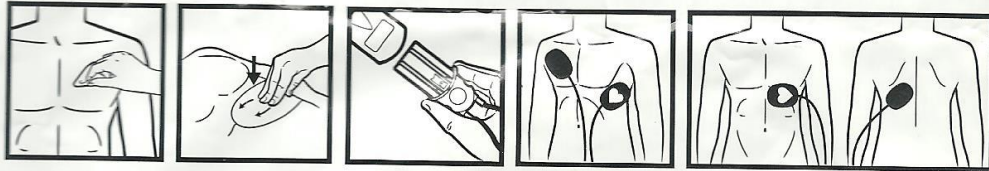
Adhere

Connect

Figure A
Anterior-Lateral Placement

Figure B
Anterior-Posterior Placement

Instructions:

1. Remove excessive hair.
2. Clean and dry skin sites.
Note: do not use alcohol or tincture of benzoin.
3. Remove electrodes from package. Separate leadwires.
4. Remove the electrodes' protective liner.
5. Position electrodes as pictured in Figure A or Figure B. Do not place on broken skin, if possible. Figure A and Figure B are guidelines for positioning electrodes and are not intended to replace clinical preferences.
6. Smooth the electrode from the center outward to the edges with fingertips to ensure that there are no air pockets between the gel and the patient's skin.
7. Electrodes are not repositionable. Replace with new electrodes if repositioning is required.
This will ensure optimal adhesion and minimize potential patient skin burns.
8. Connect the electrodes to the defibrillator or multi-function cable.
9. Follow the defibrillator instructions.
10. Replace the electrode pads after 24 hours on skin or 50 defibrillation shocks. See Instructions for Use to determine recommended pacing time. (Note: Maximum pacing duration is dependent upon pacing rate, current, and pacer waveform.)
11. Remove electrode by slowly peeling pad from the patient's skin. Discard used electrodes.

Warnings:

- Place electrodes according to defibrillator manufacturer's operating instructions.
- Do not discharge hand-held paddles through these electrodes.
- Do not use if electrode or pouch is damaged.
- Do not open package until immediately prior to use.
- Important: Replace the electrode pads after 24 hours on skin or 50 defibrillation shocks. See Instructions for Use to determine recommended pacing time. (Note: Maximum pacing duration is dependent upon pacing rate, current, and pacer waveform.)

Important:

- Operator should be thoroughly familiar with electrode operating instructions prior to use.
- This product is designed for patients >10 kg (22 lbs).
- Use separate ECG electrodes when performing noninvasive pacing.
- Do not crush, fold, or store under heavy objects.
- This product is not sterile.

Can be used in place of Koninklijke Philips Electronics N.V. M3713A, M3716A, M3718A, DP2 & DP6 electrodes
 Peut être utilisées à la place des électrodes Koninklijke Philips Electronics N.V. M3713A, M3716A, M3718A, DP2 et DP6
 Kompatibel zu Elektroden von Koninklijke Philips Electronics N.V., M3713A, M3716A, M3718A, DP2 und DP6
 Possono essere utilizzati al posto degli elettrodi M3713A, M3716A, M3718A, DP2 e DP6 Koninklijke Philips Electronics N.V.
 Pueden utilizarse en lugar de los electrodos M3713A, M3716A, M3718A, DP2 y DP6 de Koninklijke Philips Electronics N.V.
 Kan användas istället för elektroder från Koninklijke Philips Electronics N.V. med nummer M3713A, M3716A, M3718A, DP2 och DP6
 Kunnen worden gebruikt in plaats van Koninklijke Philips Electronics N.V. M3713A, M3716A, M3718A, DP2 & DP6 elektroden
 Podem ser utilizados em vez de electrodos M3713A, M3716A, M3718A, DP2 e DP6 da Koninklijke Philips Electronics N.V.
 Voidaan käyttää Koninklijke Philips Electronics N.V:n elektrodien M3713A, M3716A, M3718A, DP2 ja DP6 sijaan
 Kan användes i stället för Koninklijke Philips Electronics N.V. M3713A, M3716A, M3718A, DP2- og DP6-elektroder
 Μπορούν να χρησιμοποιηθούν στη θέση των ηλεκτροδίων M3713A, M3716A, M3718A, DP2 & DP6 της Koninklijke Philips Electronics N.V.
 Lze je použít místo elektrod společnosti Koninklijke Philips Electronics N.V. M3713A, M3716A, M3718A, DP2 & DP6
 A Koninklijke Philips Electronics N.V. M3713A, M3716A, M3718A, DP2 és DP6 típusú elektrodái helyett használhatók
 Можно использовать вместо электродов производства компании Koninklijke Philips Electronics N.V. (номер по каталогу M3713A, M3716A, M3718A, DP2 и DP6).

LOT 301841 X

2015-01

01864

U.S. Patents 5,571,165; 5,733,324; 5,824,033; 6,600,957.

© 2012 Covidien. Made in USA.

Covidien Inc, 15 Hampshire Street, Mansfield, MA 02048 USA.

Covidien Ireland Limited, IDA Business & Technology Park, Tullamore.

CH42P62662

(01)20884527022349

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